Creating balance in healthcare

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Active standup aid

**Important:**

Read these instructions carefully before using the Nemo!

Date first used .....

User .....

Name .....

Address .....

**Manufacturer**

Lopital Nederland B.V.

Laarakkerweg 9

5061 JR Oisterwijk

The Netherlands

T +31 (0)13 5239300

F +31 (0)13 5239301

www.lopital.com

info@lopital.nl

All of the aids produced by Lopital fulfil the following norms:

MDD 93/42 EEG, NEN-EN-ISO 14971, NEN-EN 12183, ISO 17966, ISO 9001:2015, ISO 13485:2016/ C12:2018, NEN-EN 12182, ISO 60601, EN ISO 60601-1-2, EN ISO 60601-1-6, EN ISO 62366, CAN/CSA 60601, ANSI/AAMI ES 60601-1

Notes on symbols	
	Read the instructions for use before use.
	Note.

# Contents

1.	Introduction .....	17
1.1.	Introduction .....	17
1.2.	Safety.....	17
1.3.	Warranty.....	19
2.	Use.....	19
2.1.	First use.....	19
2.2.	Working with the Lotus.....	20
3.	Operation .....	21
3.1.	Wheel blocking.....	21
3.2.	Base spreading.....	21
3.3.	Rotatable sitting parts.....	22
4.	Cleaning and maintenance .....	22
5.	Storage and transport.....	24
6.	Removal of parts.....	24
7.	Technical specifications .....	24
7.1.	Structure.....	24
7.2.	Dimensions .....	24

# 1. Introduction

## 1.1. Introduction

The Lotus is an active standup aid for persons that can still stand upright independently and have sufficient trunk stability but cannot or find it difficult to walk. The carer can move the client safely and efficiently after he / she moved onto the Lotus. The Lotus is meant for short transfers, for instance from the bed to the toilet. Because the client takes on an active sitting position, less force is needed to return to the standing position.

Comfort is the primary consideration for both the client and the carer. The Lotus is fitted with rotatable sitting parts, flexible lower-leg supports and base spreader. The base spreader ensures that the Lotus can be wheeled close to the client, meaning that he / she can stand up effortlessly. The Lotus stimulates the client to be more active and to assist during transfers.

The average technical life-cycle of the Lotus is 10 years, assuming it is cleaned daily and regularly maintained according to the maintenance schedule supplied (*See chapter 4. Cleaning and maintenance*).

*The Lotus is a product with CE-marking. This marking corresponds with directive 93/42/EEG and the revised directive 2007/47/EG for medical devices of 21 March 2010 and international standard ISO10535.*

## 1.2. Safety

**A failure to comply with the following safety points and the further definitions in these instructions for use could lead to hazardous situations. Please read carefully before use.**

### *Use*

- Make sure you always keep in contact with the client when using the Lotus.
- Make sure that the sitting parts are lowered before the client takes place and you are about to move the Lotus.
- Make sure that the client always holds the handles when you intend to move the Lotus.
- The client must sit on both halves; when sitting on only one part of the seat this can lead to instability.
- Make sure that the client always keeps his / her feet well-positioned on the foot support plate when using the Lotus.

- Notify the client before moving the Lotus.
- Be aware of obstacles when moving the Lotus.
- It is not permitted to charge the Lotus with more than 200 kg. / 440 lbs.
- The Lotus may not be used to transport loads other than people.
- Only use original parts, supplied by Lopital.
- Use the Lotus in a clean and tidy working environment.
- Lotus can be safely used in wet area (for example a bathroom).
- Swimming pool locations are not considered to be 'normal conditions' and will substantially shorten the lifespan of certain components.
- Only verifiably authorised personnel is allowed to work with the Lotus.
- The Lotus cannot be used on a surface with a gradient over 5 degrees laterally.
- The Lotus cannot be used on a surface with a gradient of over 10 degrees forwards / backwards.
- The Lotus is not a shower appliance.
- The Lotus may only be used indoors

#### *Maintenance*

- The Lotus must be inspected and serviced at least once a year.
- Servicing and maintenance of the Lotus may only be carried out by persons authorised by Lopital.
- Constructive components of the Lotus may only be replaced by suitably authorised Lopital personnel.
- If erroneous use, transport, an accident or improper maintenance leads to deformation of the Lotus, it may no longer be used and you must contact the supplier.

In the event of breakdowns, you must contact Lopital or your local supplier.

E-mail: [service@lopital.nl](mailto:service@lopital.nl)

**The Lotus must only be operated by persons who have been sufficiently trained and expert personnel who have experience working with the target group.**

**The Lotus must be used in a clinical setting.**

**Making changes to the construction could affect the safety of the Lotus. This will also lead to Lopital's liability and warranty conditions being invalidated. As a result, the Lotus will no longer comply with the guidelines for medical devices.**

### 1.3. Warranty

All Lopital products are supplied with a 2-year warranty, on construction and/or material faults, unless otherwise stated. In the event of varying warranty terms, the exact period will be indicated on your invoice; this also serves as the warranty itself. Components and batteries that are subject to wear and tear will not fall under warranty unless there are clear construction and / or material faults.

## 2. Use

### 2.1. First use

Before you use the Lotus, you must check to ensure the following components are present. These components can be replaced by authorised individuals in the event of damage. These components are the applied parts of the Lotus. These components may come into contact with the user(s) while the Lotus is in use.

## 2.2. Working with the Lotus

### *Step 1*

Roll the Lotus towards the client and rotate the sitting parts away (*See paragraph 3.3. Rotatable sitting parts*).

### *Step 2*

Spread if necessary the base parts of the Lotus and roll it as close as possible to the client. Have the client put his / her feet on the footrests. Make sure that the shins touch the flexible lower leg supports (*See paragraph 3.2. Base spreading*).

### *Step 3*

The Lotus must be put on the brakes before transferring persons (*See paragraph 3.1. Wheel blocking*).

### *Step 4*

The client can now stand upright. The client can take hold of the vertical hand grips to lever himself / herself into a standing position.

### *Step 5*

Turn the sitting parts away. Now the client can take place on the Lotus.

### *Step 6*

Disengage the breaks if you wish to move the Lotus (*See paragraph 3.1. Wheel brakes*).

### *Step 7*

If wanted, you can move the base spreader to the narrower position by pushing the other pedal. Roll the client towards the (wheel)chair, toilet or bed (*See paragraph 3.2. Base spreading*).

### *Step 8*

Spread if necessary the base parts of the Lotus and roll it as far as possible to the (wheel) chair, toilet or bed.

### *Step 9*

The Lotus must be put on the brakes before transferring persons (*See paragraph 3.1. Wheel blocking*).

**Step 10**

Have the client stand upright. The client can take hold of the vertical handgrips to pull himself / herself into a standing position.

**Step 11**

Turn the sitting parts away again. Now the client can take place on the (wheel)chair, toilet or bed.

**Step 12**

Have the client take his / her feet from the footplate. Disengage the brake for moving the Lotus.

## 3. Operation

### 3.1. Wheel blocking

The Lotus has 2 braked wheels on the side of the caretaker. The 2 smaller swivel wheels are unrestrained.

- The brake can be applied on the Lotus by pushing the blue lip (1) on the wheel downwards (See figure 1).
- In order to take the Lotus off the brake, the same lip on the wheel must be moved upwards by tapping the upper side of the lip (2) (See figure 1).

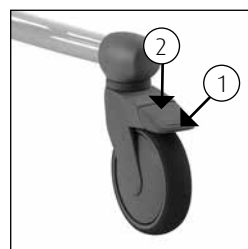
Figure 1 Swivel wheel.

### 3.2. Base spreading

The Lotus can be used as a standup aid with shower-toilet chairs, shower stretchers, beds, toilets, chairs and wheel-chairs. The Lotus is equipped with a mechanical base spreader. By using the base spreader, the Lotus can be rolled close to the person with the disability. In the narrower position, the Lotus can be manoeuvred easily in small rooms. By kicking one of the pedals, the base parts will move inwards or outwards (depending on the pedal chosen).

Figure 2 Base spreader.

### 3.3. Rotatable sitting parts

The Lotus is equipped with 2 comfortable sitting parts that can be rotated away. Rotate both parts away before rolling the Lotus towards the client (See figure 3). Have the client stand on the Lotus. Ensure that he / she positions his / her legs in the centre of the footplate and that the shins rest against the lower leg support. Now the client can pull himself / herself up. Now, rotate the sitting parts back, thus enabling the client to sit down (See figure 4).

Figure 3 Sitting parts rotated away.

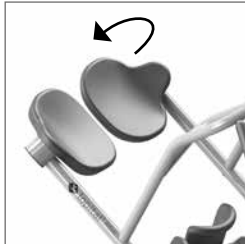
Figure 4 Sitting parts.

## 4. Cleaning and maintenance

The Lotus must be cleaned after every use, with water and regular household cleaning agents. PUR integrated foam components with IMC coating should be rubbed clean and disinfected. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol.

The time the products must be left on the components has a significant impact upon the lifespan of the PUR components. As a result of the material properties of PUR, contact with sharp objects must be avoided. This could lead to damage or tears in the material. Finally, we would like to point out that non-critical parts of the PUR components must also be cleaned and disinfected.

Check the wheels regularly for hairs, dust and soap residues and remove when necessary. Maintenance and repairs to the Lotus may only be carried out by authorised personnel.

<b>Maintenance schedule</b>	
<b>After every use</b>	<ul style="list-style-type: none"> <li>• Clean the <b>Lotus</b> with water and regular household cleaning products.</li> <li>• PUR foam components must be cleaned and disinfected.</li> </ul>
<b>Weekly</b>	<ul style="list-style-type: none"> <li>• Clean frame.</li> <li>• Check wheels for hairs, dust and soap residues and remove if necessary.</li> <li>• Check that the <b>Lotus</b> is still working correctly.</li> </ul>
<b>Annually</b>	<ul style="list-style-type: none"> <li>• According to the guidelines for medical devices, the <b>Lotus</b> must be maintained in line with the manufacturer's instructions.</li> </ul>

## 5. Storage and transport

During transport and storage, the ambient conditions must remain moderate. In other words, the temperature should remain between -40°C and +55°C (-40°F to 130°F) and humidity levels should be kept as low as possible. Vibration during transport must be kept to a minimum in order to avoid damaging the hinged mechanisms on the Lotus.

## 6. Removal of parts

All parts that are replaced or removed can be sent back to Lopital BV. You may also offer the parts to the engineer for destruction. We will ensure that all parts are processed in an environmentally-friendly manner.

## 7. Technical specifications

The Lotus is splash-proof. This is indicated by the IPX4 mark.

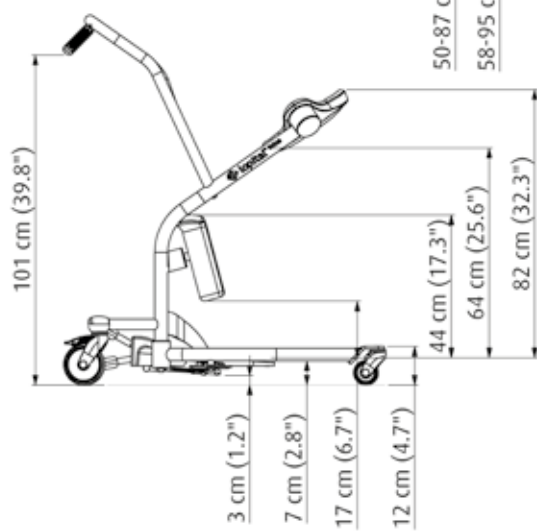
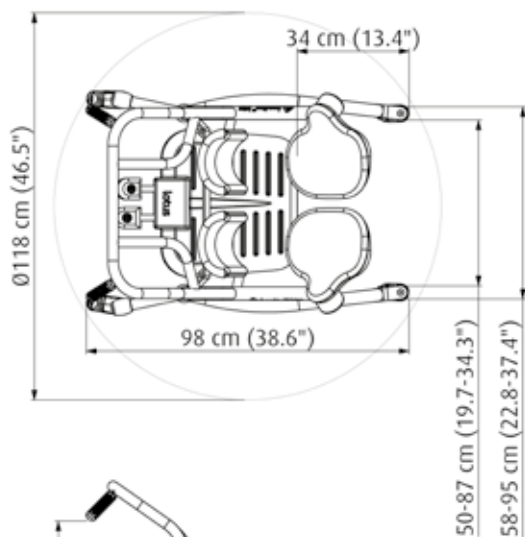
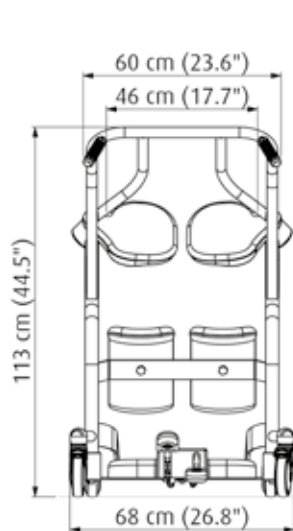
### 7.1. Structure

Frame:	Stainless steel and steel tubes and plates that are equipped with a two-layer powder coating
Foot plates:	Plastic V0
Weight:	35 kg / 77.2 lbs
Maximum permitted load:	200 kg / 440 lbs

### 7.2. Dimensions

Surface area:	Wide:	68 cm / 26.8 inch narrow position
		95 cm / 37.4 inch with base spread out
	Deep:	98 cm / 38.6 inch

**Notes**

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

# Mode d'emploi

## Lotus

5900 8100

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 **lopital**<sup>®</sup>  
Creating balance in healthcare

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Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk