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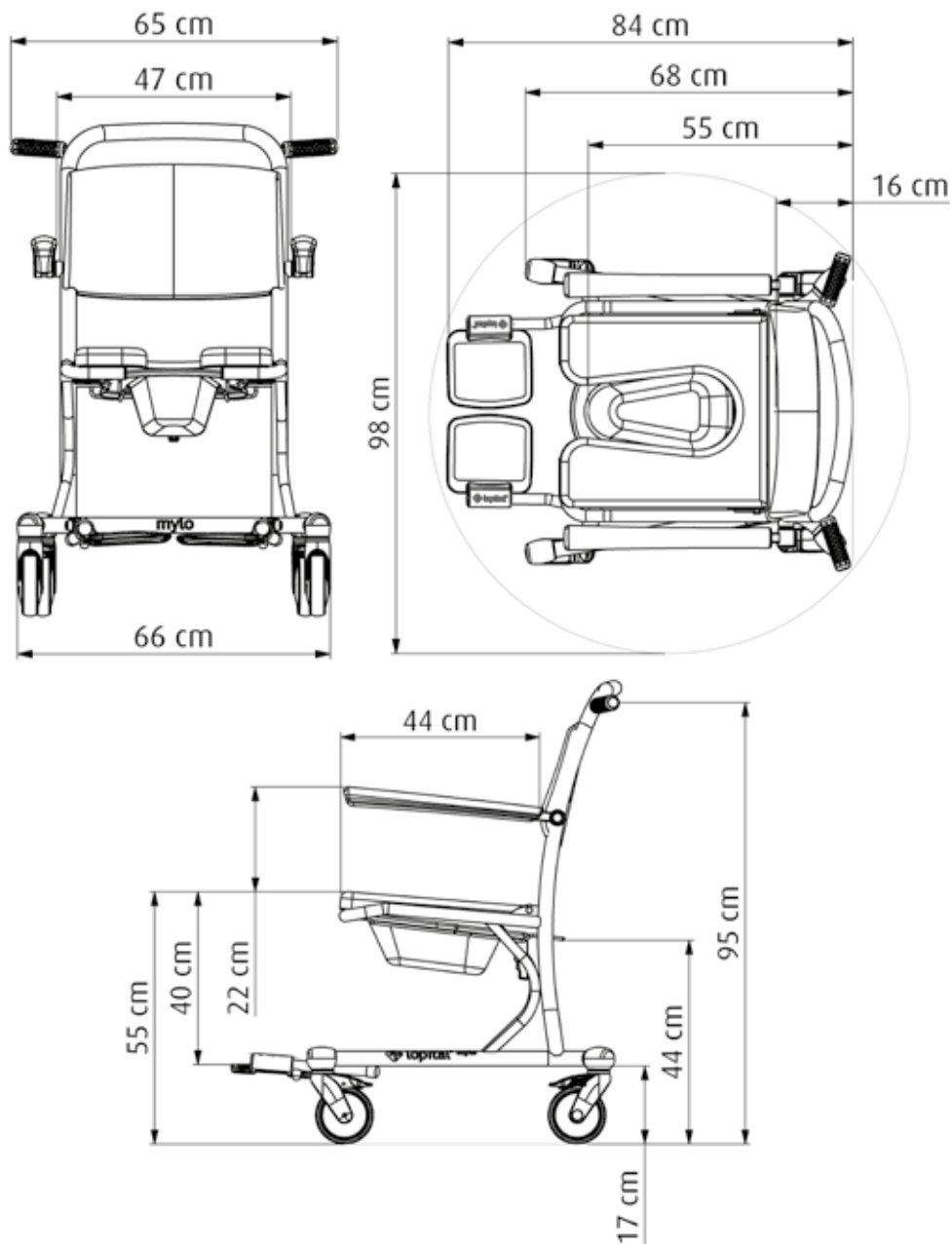
Shower-toilet chair

Important:

Read these instructions carefully before using the Mylo!

Date first used

User

Name

Address

Manufacturer

Lopital Nederland B.V.

Laarakkerweg 9

5061 JR Oisterwijk

The Netherlands

T +31 (0)13 5239300

F +31 (0)13 5239301

www.lopital.com

info@lopital.nl

All of the aids produced by Lopital fulfil the following norms:

MDD 93/42 EEG, NEN-EN-ISO 14971, NEN-EN 12183, ISO 17966, ISO 9001:2015, ISO 13485:2016/ C12:2018, NEN-EN 12182, ISO 60601, EN ISO 60601-1-2, EN ISO 60601-1-6, EN ISO 62366, CAN/CSA 60601, ANSI/AAMI ES 60601-1

Notes on symbols	
	Read the instructions for use before use.
	Note.

Contents

1.	Introduction	19
1.1.	Introduction	19
1.2.	Safety.....	19
1.3.	Warranty.....	20
2.	Use.....	21
2.1.	First use.....	21
2.2.	Working with the Mylo.....	22
3.	Operation	23
3.1.	Wheel blocking.....	23
3.2.	Footrests.....	23
3.3.	Armrests.....	23
3.4.	Toilet function.....	24
3.4.1.	In combination with toilet.....	24
3.4.2.	Use of a bedpan or bucket.....	24
3.5.	Foldable and removable seat.....	25
4.	Cleaning and maintenance	26
5.	Storage and transport.....	27
6.	Removal of parts.....	28
7.	Technical specifications	28
7.1.	Structure.....	28
7.2.	Dimensions	28

1. Introduction

1.1. Introduction

The shower-toilet chair Mylo is a medical device for caring for and washing clients in an ergonomically responsible manner. Comfort is the primary consideration for both the client and the carer.

The Mylo can also be used as a toilet chair, in combination with a bedpan / bucket. The Mylo is a shower-toilet chair with independent, folding armrests, polyurethane seat, rails for a bedpan and rotating and foldable footrests. Furthermore, Mylo is manufactured with a foldable and removable seat, making it easy to clean the seat. These functions are essential performance.

The average technical life-cycle of the Mylo is 10 years, assuming it is cleaned daily and regularly maintained according to the maintenance schedule supplied (See chapter 4. *Cleaning and maintenance*).

The Mylo is a class 1 product with CE-marking. This marking corresponds with directive 93/42/EC and the revised directive 2007/47/EC for medical devices of 21 March 2010.

1.2. Safety

A failure to comply with the following safety points and the further definitions in these instructions for use could lead to hazardous situations. Please read carefully before use.

Use

- Make sure that the Mylo brake is applied before transferring clients.
- Make sure that the Mylo seat is always folded down before transferring clients.
- Notify the client before moving the Mylo.
- Make sure that the client always places his / her arms on the appropriate armrest in order to avoid anything becoming jammed.
- Keep the armrests down / closed when moving the Mylo.
- Always try to move backwards over obstacles (rather than forwards).
- It is not permitted to charge the Mylo with more than 150 kg / 330 lbs.
- The Mylo may not be used to transport loads other than people.
- Only use original parts, supplied by Lopital.
- Use the Mylo in a clean and tidy working environment.
- Only verifiably authorised personnel is allowed to work with the Mylo.

- Swimming pool locations are not considered to be ‘normal conditions’ and will substantially shorten the lifespan of certain components.
- The Mylo cannot be used on a surface with a gradient over 5 degrees laterally.
- The Mylo cannot be used on a surface with a gradient of over 10 degrees forwards / backwards.
- During use of the Mylo, the water temperature may not exceed 43°C / 109.4°F.
- The Mylo may only be used indoors.

Maintenance

- The Mylo must be inspected and serviced at least once a year.
- Servicing and maintenance of the Mylo may only be carried out by persons authorised by Lopital.
- Constructive components of the Mylo may only be replaced by suitably authorised Lopital personnel.
- If erroneous use, transport, an accident or improper maintenance leads to deformation of the Mylo, it may no longer be used and you must contact the supplier.

In the event of breakdowns, you must contact Lopital or your local supplier.

E-mail: service@lopital.nl

The Mylo must only be operated by persons who have been sufficiently trained and expert personnel who have experience working with the target group.

The Mylo must be used in a clinical setting.

Making changes to the construction could affect the safety of the Mylo. This will also lead to Lopital’s liability and warranty conditions being invalidated. As a result, the Mylo will no longer comply with the guidelines for medical devices.

1.3. Warranty

All Lopital products are supplied with a 2-year warranty, on construction and/or material faults, unless otherwise stated. In the event of varying warranty terms, the exact period will be indicated on your invoice; this also serves as the warranty itself.

Components and batteries that are subject to wear and tear will not fall under warranty unless there are clear construction and / or material faults.

2. Use

2.1. First use

Before you use the Mylo, you must check to ensure the following components are present. These components can be replaced by authorised individuals in the event of damage. These components are the applied parts of the Mylo. These components may come into contact with the user(s) while the Mylo is in use.

2.2. Working with the Mylo

Step 1

If necessary, place the seat once again on the chair and / or fold the seat downwards.

Step 2

If necessary, fold the armrests up (*See paragraph 3.3. Armrests*).

Step 3

If necessary, fold the foot plates up and fold the footrest frame up too (*See paragraph 3.2. Footrests*).

Step 4

The Mylo must be put on the brakes before transferring persons (*See paragraph 3.1. Wheel blocking*).

Step 5

The client can take a seat in the chair. Support the client if necessary.

Step 6

Place both feet on the footrests (*See paragraph 3.2. Footrests*).

Step 7

Close the armrests once the client has taken a seat in the chair (*See paragraph 3.3. Armrests*).

Step 8

Notify the client about the processes you will be carrying out.

Step 9

Take the chair off the brake when you wish to move it (*See paragraph 3.1. Wheel blocking*).

3. Operation

3.1. Wheel blocking

- The brake can be applied on the Mylo by pushing the blue lip (1) on the wheel downwards (See figure 1).
- In order to take the Mylo off the brake, the same lip on the wheel must be moved upwards by tapping the upper side of the lip (2) (See figure 1).

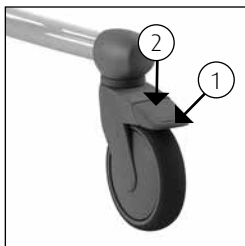
Figure 1 Swivel wheel.

Figure 2 Folding up footrests.

Option: Directional wheel left front
(Article number 7556 0025).

3.2. Footrests

- Before the client takes a seat in the Mylo, the foot plates or footrest frame must be folded upwards. There is now sufficient room to allow the client to easily take a seat (See figures 2 & 3).
- The carer or the client first raises one leg; the carer then folds the foot plate down (See figure 4). The same procedure is then carried out with the other leg.

Figure 3 Folding up footrest frame.

Figure 4 Folding down footrests.

3.3. Armrests

- The independent folding armrests are easy to fold up.
- Protect the user when moving the armrests upwards and downwards.
- In the folded up position, they should not fall back down by themselves; in other cases: consult your supplier.

When folding the armrests away, avoid the hinging point so as to avoid becoming jammed.

Figure 5 Armrest Mylo.

When folding the armrests away, avoid fingers or body parts becoming jammed between the armrests and the back support.

The armrests or front closure must not be used as a lever arm (e.g. when moving the chair over an obstacle) or as a push / pull bar. You must not sit on it either. This could damage the Mylo and / or present a hazard.

3.4. Toilet function

3.4.1. In combination with toilet

- The Mylo can now be moved backwards over the toilet.

The Mylo does not fit over every toilet.

3.4.2. Use of a bedpan or bucket

- If the Mylo is being used as a toilet chair, the special guides under the seat can be used to accommodate a stainless steel / plastic bedpan or bucket.
- The bedpan or bucket is slid in from the back of the chair (See figure 6).
- If you are using the Lopital plastic bedpan, you must slide it as far forwards as possible (See figure 7).
- If you are using a stainless steel bedpan or bucket, the bedpan can be placed in 2 positions under the seat. At the front of the seat (See figure 8) or at the back of the seat (See figure 9).

Figure 6 Bedpan.

Figure 7 Lopital plastic bedpan.

Figure 8 Stainless steel bedpan, front.

Figure 9 Stainless steel bedpan, back.

In order to avoid injuries, we advise the carer to insert the bucket or bedpan before the client takes a seat on the chair.

The Lopital plastic bedpan, stainless steel bedpan and bucket must be cleaned and disinfected after use, for example in a bedpan washing unit.

If the Lopital plastic bedpan, stainless steel bedpan or bucket has been used, it must be removed from the chair before the chair is moved or adjusted in height.

3.5. Foldable and removable seat

If required, you can remove the Mylo seat from the frame. That makes it much easier to clean it and no dirt will remain between the frame and the seat. The seat can be removed from the frame without the need for any tools (See figure 10 & 11).

Figure 10 Folding the seat.

Figure 11 Removing the seat.

4. Cleaning and maintenance

The Mylo must be cleaned after every use, with water and regular household cleaning agents. PUR integrated foam components with IMC coating should be rubbed clean and disinfected. Do not use any abrasive or other aggressive cleaning products. If necessary, the PUR components can be disinfected with a solution of 70% ethanol or propanol.

The time the products must be left on the components has a significant impact upon the lifespan of the PUR components. As a result of the material properties of PUR, contact with sharp objects must be avoided. This could lead to damage or tears in the material. Finally, we would like to point out that non-critical parts of the PUR components must also be cleaned and disinfected.

Check the wheels regularly for hairs, dust and soap residues and remove when necessary. Maintenance and repairs to the Mylo may only be carried out by authorised personnel.

Maintenance schedule	
After every use	<ul style="list-style-type: none"> • Clean chair with water and regular household cleaning products. • PUR foam components must be cleaned and disinfected. • Check to ensure all parts of the Mylo are present.
Weekly	<ul style="list-style-type: none"> • Clean frame. • Check wheels for hairs, dust and soap residues and remove if necessary. • Check that the chair is still working correctly.
Annually	<ul style="list-style-type: none"> • According to the guidelines for medical devices, a chair must be maintained in line with the manufacturer's instructions.

5. Storage and transport

During transport and storage, the ambient conditions must remain moderate. In other words, the temperature should remain between -40°C and +55°C (-40°F to 130°F) and humidity levels should be kept as low as possible. Vibration during transport must be kept to a minimum in order to avoid damaging the hinged mechanisms on the chair.

6. Removal of parts

All parts that are replaced or removed can be sent back to Lopital BV. You may also offer the parts to the engineer for destruction. We will ensure that all parts are processed in an environmentally-friendly manner.

7. Technical specifications




The Mylo is splash-proof. This is indicated by the IPX4 mark.

7.1. Structure

Frame:	Stainless steel and steel tubes and plates that are equipped with a 2-layer powder coating
Hinge points:	Stainless steel, cover caps made of plastic V0
Wheels:	Plastic, Ø 12.5 cm / 4.9 inch, double blocking facility
Body support:	PUR (polyurethane).
Weight:	19 kg / 41.9 lbs
Maximum permitted load:	150 kg / 330 lbs

7.2. Dimensions

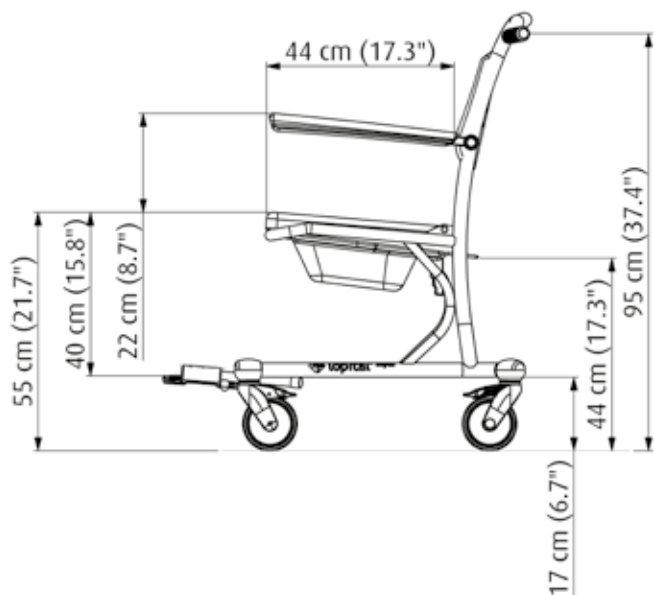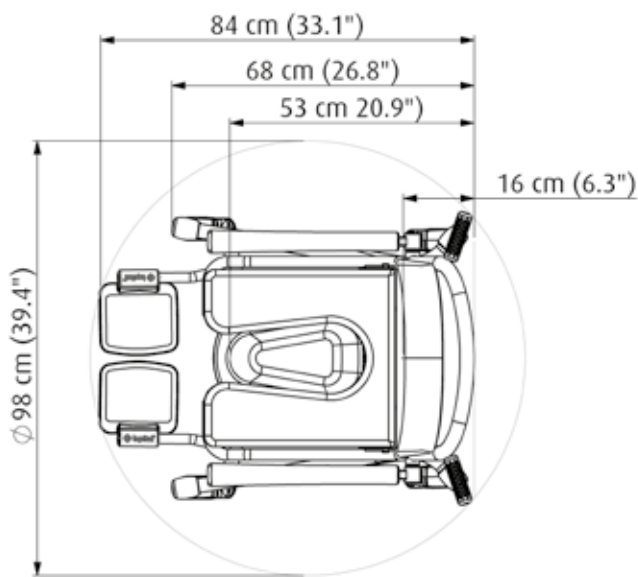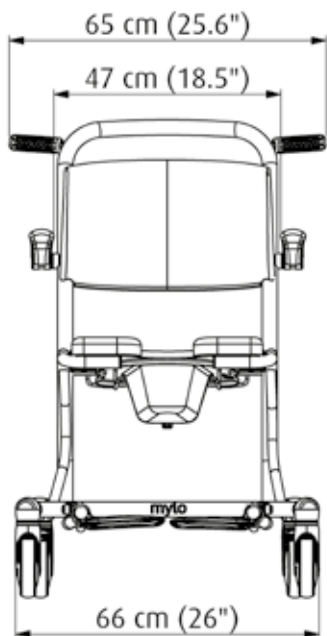Surface area:	66 cm / 26 inch wide, 84 cm / 33.1 inch deep
Seat height:	55 cm / 21.7 inch

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Lopital Nederland B.V. Laarakkerweg 9, 5061 JR Oisterwijk, Postbus 56, 5060 AB Oisterwijk