


P:\SW-Tek\Vienna\Vienna 5030\

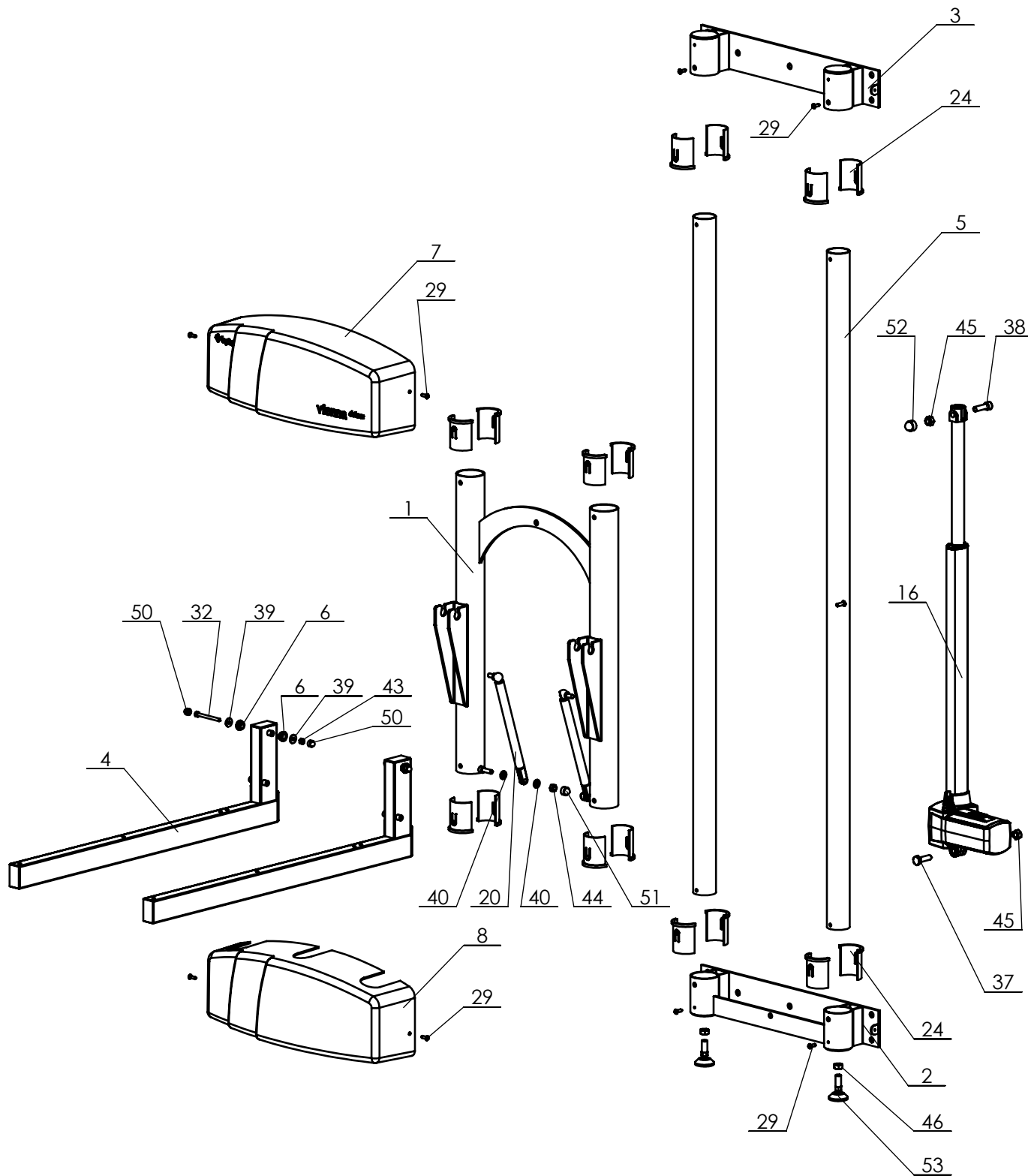

|           |       |      |            |
|-----------|-------|------|------------|
| Drawn by: | RH    | Date | 11-05-2023 |
| Revised   | 1 /KA | Date | 11-11-2025 |
| Revised   | - /KA | Date | -          |


Copyright reserved  
by law

Unit of measure-  
ment: mm

## Overview parts Vienna deluxe wall

Lopital Nederland bv

Draw. no.: EXPV5030-2025-01

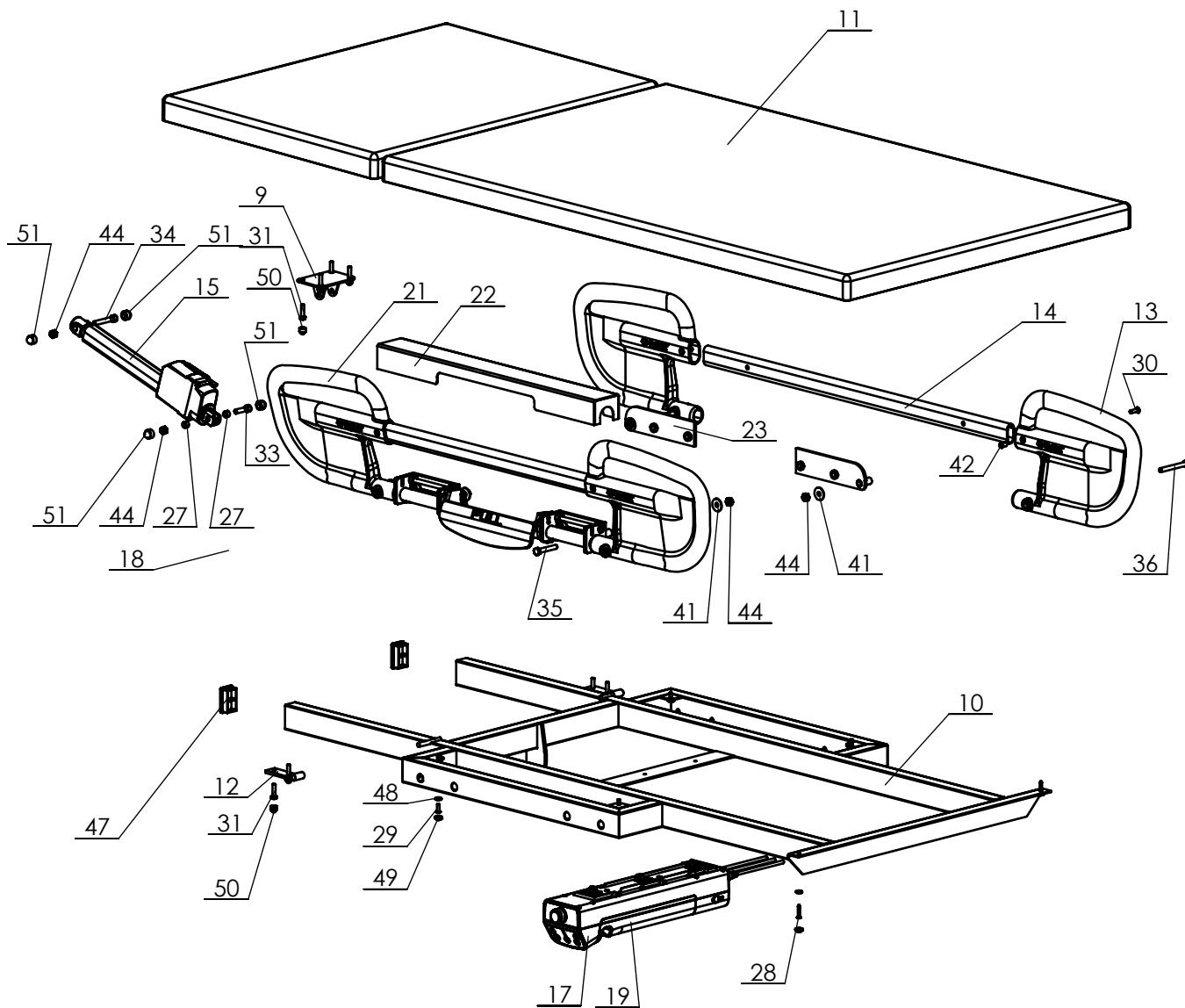

P:\SW-Tek\Vienna\Vienna 5030\

|           |       |      |            |
|-----------|-------|------|------------|
| Drawn by: | RH    | Date | 11-05-2023 |
| Revised   | 1 /KA | Date | 11-11-2025 |
| Revised   | - /KA | Date | -          |

Copyright reserved  
by law

Unit of measure-  
ment: mm

## Overview parts Vienna deluxe wall


Lopital Nederland bv

Draw. no.: EXPV5030-2025-01

| Vienna Deluxe wall-mounted dressing table, Article no: 6500 5030 |          |                         |                                   |                                |
|------------------------------------------------------------------|----------|-------------------------|-----------------------------------|--------------------------------|
| Part no.                                                         | Art.no.  | QTY.                    | Name                              | Drawing no.                    |
| 1                                                                | 70250001 | 1                       | Movable frame                     | 2500-13                        |
| 2                                                                | 70250002 | 1                       | Floor frame                       | 2500-14                        |
| 3                                                                | 70250003 | 1                       | Wall frame                        | 2500-15                        |
| 4                                                                | 70250004 | 2                       | Support arm                       | 2500-12                        |
| 5                                                                | 70250007 | 2                       | Guide tube                        | 2500-17                        |
| 6                                                                | 70250008 | 4                       | Bearing bush brass Ø16x11,2x8 mm  | 2500-16                        |
| 7                                                                | 70250015 | 1                       | Cover upper wall frame ABS        | 2500-20                        |
| 8                                                                | 70250016 | 1                       | Cover lower wall frame ABS        | 2500-19                        |
| 9                                                                | 70503005 | 1                       | Bracket Backrest adjustment       | 5030-14.1                      |
| 10                                                               | 70200004 | 1                       | Top frame                         | 5030-11.1                      |
| 11                                                               | 70503006 | 1                       | Mattress 2-piece 190x79           | 5030-17                        |
| 12                                                               | 70402507 | 2                       | Hinge backrest adjustment         | 4110-14                        |
| 13                                                               | 70200026 | 2                       | Handle side rail/push bar         | 2000-35.3                      |
| 14                                                               | 70200019 | 1                       | Oval tube side rail               | 2000-28.2                      |
| 15                                                               | 70205005 | 1                       | Actuator TA29 backrest adjustment | TA29                           |
| 16                                                               | 70250011 | 1                       | Actuator TA37 high/Low            | TA37-Slag 500                  |
| 17                                                               | 70205002 | 1                       | Control box TC12                  | 3D_TC12 (2.9A)                 |
| 18                                                               | 70205004 | 1                       | Handcontrol                       | 3D_TMH3                        |
| 19                                                               | 70205003 | 1                       | Battery box 2.9Ah                 | Accu                           |
| 20                                                               | 70250018 | 2                       | Gas spring + ball joint radial    | 150mm 700N                     |
| 21                                                               | 67902020 | 1                       | Side rail compleet                | 2000-ZH                        |
| 22                                                               | 70503002 | 1                       | Cover side rail                   | 5030-13                        |
| 23                                                               | 70503003 | 1                       | Bracket set side rail             | 5030-12.1                      |
| 24                                                               | 70250010 | 16                      | Bearing shell POM/PTFE            | 2500-21                        |
| 25                                                               |          | 1                       | Sticker Vienna deluxe large       | Sticker - Vienna deluxe        |
| 26                                                               |          | 1                       | Sticker Lopital                   | Sticker - Lopital + Logo 12 mm |
| 27                                                               |          | 2                       | Bush PA 12x8,2x7,5                | 8x12x7,5                       |
| 28                                                               |          | 2                       | Chipboard screw 4,5x25            | ST4,5x25 DIN7981               |
| 29                                                               |          | 12                      | Screw M5x16 ISO7380 A2            | M5x16 ISO7380                  |
| 30                                                               |          | 2                       | Screw M6x20 DIN966 A2             | M6x20 DIN964                   |
| 31                                                               |          | 8                       | Screw M6x25 DIN931 A2             | M6x25 DIN931                   |
| 32                                                               |          | 2                       | Screw M6x65 DIN931 A2             | M6x65 DIN931                   |
| 33                                                               |          | 1                       | Screw M8x30 DIN912 A4             | M8x30 DIN912                   |
| 34                                                               |          | 1                       | Screw M8x50 DIN912 A2             | M8x50 DIN912                   |
| 35                                                               |          | 4                       | Bolt M8x50 DIN933 A2              | M8x50 DIN933                   |
| 36                                                               |          | 2                       | Bolt M8x70 DIN933 A2              | M8x70 DIN933                   |
| 37                                                               |          | 1                       | Bolt M10x35 DIN931 A2             | M10x35 DIN931                  |
| 38                                                               |          | 1                       | Screw M10x35 DIN912 A2            | M10x35 DIN912                  |
| 39                                                               |          | 4                       | Washer M6 Ø18 DIN9021 A2          | Carrosseriering M6 DIN9021     |
| 40                                                               |          | 4                       | Washer M8 DIN125 PA               | M8 DIN 125 PA                  |
| 41                                                               |          | 6                       | Washer M8 Ø25 A2                  | Carrosseriering M8             |
| 42                                                               |          | 2                       | Sleeve nut M6x10x15 A2            | M6 x10 x15                     |
| 43                                                               |          | 2                       | Locknut M6 DIN985 A2              | M6 DIN985                      |
| 44                                                               |          | 10                      | Locknut M8 DIN985 A2              | M8 DIN985                      |
|                                                                  |          |                         |                                   |                                |
| Drawn by: RH    Date 11-05-2023                                  |          |                         | Overview parts Vienna deluxe wall |                                |
| Revised 1 /KA    Date 11-11-2025                                 |          |                         |                                   |                                |
| Revised - /KA    Date -                                          |          |                         |                                   |                                |
| Copyright reserved by law                                        |          | Unit of measurement: mm | Lopital Nederland bv              | Draw. no.: EXPV5030-2025-01    |

## Vienna Deluxe wall-mounted dressing table, Article no: 6500 5030

| Part no. | Art.no. | QTY. | Name                  | Drawing no.      |
|----------|---------|------|-----------------------|------------------|
| 45       |         | 2    | Locknut M10 DIN985 A2 | M10 DIN985       |
| 46       |         | 2    | Nut M10 DIN934 A4     | M10 A2 DIN 934   |
| 47       |         | 2    | Tube cap 50x30        | Insteedkop 50x30 |
| 48       |         | 6    | Capring M5            | Kapring M5       |
| 49       |         | 6    | Screw cap M5          | Afdekkap M5      |
| 50       |         | 12   | Nut cap M6            | Moerkap M6       |
| 51       |         | 6    | Nutcap M8             | Moerkap M8       |
| 52       |         | 1    | Nutcap M10            | Moerkap M10      |
| 53       |         | 2    | Adjustable foot M10   | Stelvoet M10     |

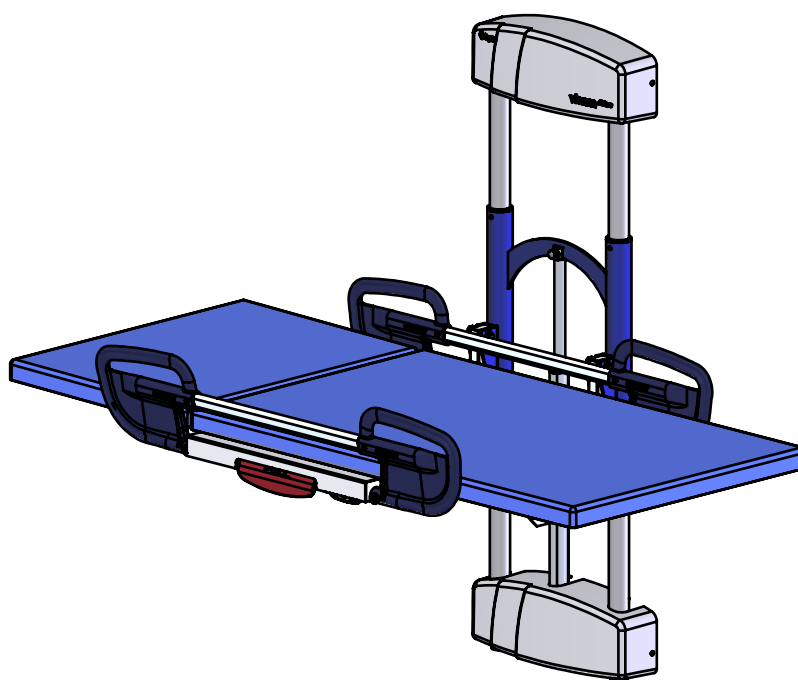

|           |       |      |            |
|-----------|-------|------|------------|
| Drawn by: | RH    | Date | 11-05-2023 |
| Revised   | 1 /KA | Date | 11-11-2025 |
| Revised   | - /KA | Date | -          |

Copyright reserved  
by law

Unit of measure-  
ment: mm

## Overview parts Vienna deluxe wall

Lopital Nederland bv

Draw. no.: EXPV5030-2025-01